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Biourine

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ALCARE

Instruction Manual

- Prior to use, carefully read the instruction manual so that you understand the product's features.
- Keep the instruction manual close at hand and read it again when needed.

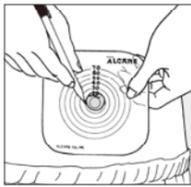
You can contact us at
E-mail: inquiry@alcare.co.jp
F a x : (8 1) 3 - 5 6 1 1 - 7 8 2 7

How to Apply Faceplate

①Measure Your Stoma Size

Measure your stoma size and make the stoma size hole in the cutting gauge. Make sure the hole comes center of the gauge. Keep the cutting gauge as your template.


* Stoma size may change. Measure once a month to determine your current stoma size.

②Make a Hole on Adhesive Part

Turn over the cutting gauge with a hole and place it on the release paper of the adhesive side. Trace the hole onto it. Cut the adhesive part 1 to 2 mm larger than the indicated line.

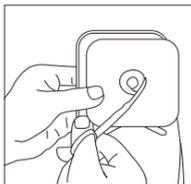* Make the inside edge of the adhesive part smooth by rubbing with fingers to avoid hurting your stoma.

③Peel Off the Release Paper

Peel the release paper off the surface of adhesive part.

④Apply the Adhesive Part

Smoothing out the wrinkle, apply the adhesive part onto the abdomen.

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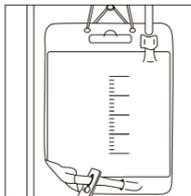
⑤Hold the Adhesive Part Well.

After application of the adhesive part, press with the hands moving outward from the peristoma to achieve firm contact with the skin.


How to Use the URINARY BAG (sold separately, ALCARE Code No.12071)


①The attached connecting tube is not needed in the case of using URINARY BAG.

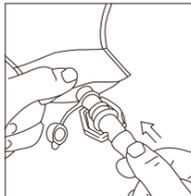②Squeeze the connector part of the URINARY BAG into the urinary outlet of pouch until you hear a click.

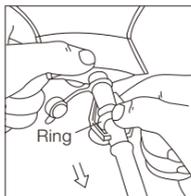③To remove the urinary bag, pull out while pressing down the ring part.

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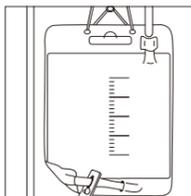
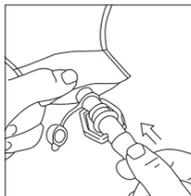
Precautions for Use and Storage

To avoid peeling off or leakage due to decreased adhesive strength, observe the following:

- Do not touch the adhesive part with oil-covered hands.
- Do not apply ointment or cream that may weaken adhesion around the stoma.
- Do not reapply an ostomy bag once it has been peeled off.
- Do not store in an area of high-temperature or high-humidity area or in a freezer.
- Do not store in direct sunlight or in a pressurized area.
- Avoid long-term stockpiling.
- Do not peel off the release paper until use.
- Discontinue use if skin becomes irritated.

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ALCARE Co., Ltd.