



SURGIDRAIN ZIPPER
Drainage after Operation

Type	Product Code No.	Specification		Contents Per Box
		Bag Size (L X W)	Max Available Size	
S	15661	158mmX110mm	30mm	10 pcs
M	15662	188mmX130mm	50mm	10 pcs
L	15663	218mmX150mm	80mmX60mm	5 pcs


EOG Sterilized




SURGIDRAIN OPEN-TOP
Drainage after Operation

Type	Product Code No.	Specification		Contents Per Box
		Bag Size (L X W)	Max Available Size	
Post-Op-M	15671	205mmX130mm	50mm	2 pcs
S	15681	175mmX110mm	30mm	2 pcs
M	15682	205mmX130mm	50mm	2 pcs
L	15683	235mmX150mm	80mmX60mm	2 pcs
LL	15684	280mmX219mm	150mmX70mm	2 pcs

EOG Sterilized

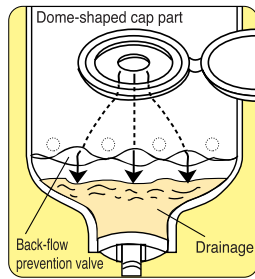


SURGIDRAIN OPEN-TOP Post -Op-M features a back-flow prevention valve.

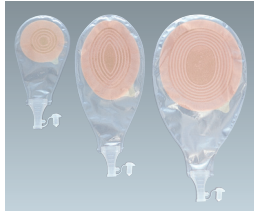

WELLCARE DRAIN
Fistula Drainage

Type	Product Code No.	Specification		Contents Per Box
		Bag Size (L X W)	Max Available Size	
S	15831	160mmX 95mm	35mm	5 pcs
M	15832	240mmX120mm	100mmX 60mm	5 pcs
L	15833	280mmX160mm	120mmX100mm	5 pcs

Accessory

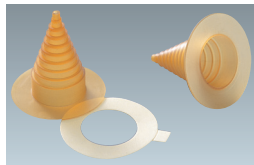


CONNECT CONE
Catheter Fixer

Type	Product Code No.	Specification (Effective Catheter Size)	Contents Per Box
—	12061	6~40Fr	3 pcs

EOG Sterilized



This appliance for fixing can be used when a continuous suction catheter or a tube is inserted in the wound. When combined with various pouches, it controls drainage efficiently and safely.

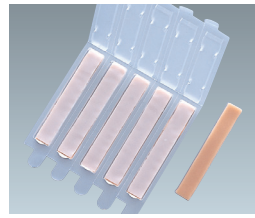

PRO CARE SOFT WAFER STICK
Wafer Skin Barrier (Karaya Gum + Citrus Pectin)

Type	Product Code No.	Specification (L X W)	Contents Per Box
Stick	15601	15mmX100mm	10 pcs



This flexible, easy-to-process skin barrier snugly fits an uneven skin surface with wrinkles and hollows to achieve close contact with the pouch.



DRAINAGE BAG
Bag for Drainage

Type	Product Code No.	Specification		Contents Per Box
		Bag Size (L X W)	Capacity	
—	15841	320mmX230mm (tube length: 1m)	2000ml	5 pcs

From Gauze to Pouch
More Efficient in Treatment
More Comfortable for Patient

During / After Operation Management Solution

Pouch Drainage System

SURGIDRAIN ZIPPER
SURGIDRAIN OPEN-TOP
WELLCARE DRAIN

ALCARE

ALCARE Co., Ltd.
INTERNATIONAL SALES DIVISION

19F, Arcacentral Bldg.
1-2-1 Kinshi, Sumida-ku
Tokyo 130-0013, Japan
TEL:81-3-5611-7819
FAX:81-3-5611-7827
<http://www.alcare.co.jp/e>

ALCARE

Innovative Solutions for
Best Care

From Gauze to Pouch

〈Pouch Drainage System〉 provides handy use in nursing and makes patients early rise.


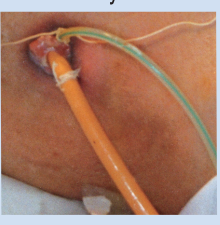


Benefits of the Pouch Drainage System

Labor Saving in Care

Comfort for Patients

- The correct amount and properties of drainage can be checked by pooling it in a pouch or drainage bag.
- Skin irritation caused by drainage attaching to the skin is reduced.
- The frequency of changing the gauze is reduced by wearing a pouch or drainage bag. This simplifies drainage care, giving patients a relaxing feeling and quiet sleep.
- The compactness and odor-tightness while wearing the pouch enable easy rising from bed.

From Gauze to Pouch ▶ Pouch Drainage System

Intended Purpose and Choice of Pouch Drainage System	What are the contents of treatment?	What are the properties of drainage?	Characteristics of the Drainage Pouch			
			Outlet Part	Skin-contacting Part	Pouch Part	
<ul style="list-style-type: none"> ● Drain Management ● Drainage Management 	From Bloody to Serous	 <p>Open Drainage</p>	<ul style="list-style-type: none"> ● Soft Touch to the Skin ● Even viscous excrement can be drained by squeezing (milking) from outside. 	<ul style="list-style-type: none"> ● Water Resistant ● Initial Adhesive Power ● Snug-Fitting 	<p>The odor proof film seals in exudate odors.</p>	<p>Observation and treatment is possible from the opening part.</p> <p>The shape snugly fits body contours.</p> <p>The shape protects wounds.</p>
<ul style="list-style-type: none"> ● Drain Management after Urostomy ● Drainage Management 	From Bloody to Serous		<ul style="list-style-type: none"> ● DRAINAGE BAG (Code 15841) is attachable. 	<p>The skin barrier offers all these superior properties.</p>	 <p>Odor Proof Film</p>	
<ul style="list-style-type: none"> ● Fistula Management 	Viscous and High pH	 <p>Fistula</p>	<p>Draining by milking</p>	<ul style="list-style-type: none"> ● Thin and Flexible ● High pH-Buffering Action 	<p>The skin barrier for fistula offers these superior properties.</p>	




SURGIDRAIN ZIPPER

The inexpensive and compact drainage pouch has an open / close zipper.

- Wounds can be observed and treated by opening the zipper, while the pouch is still attached.
- The skin barrier is soft and flexible, with little residual paste and excellent fitting.
- A soft-type outlet allows viscous excrement to be drained by milking from outside.



FLEXISTICK (for retaining the zipper opening) makes it simply to observe and treat the wound.

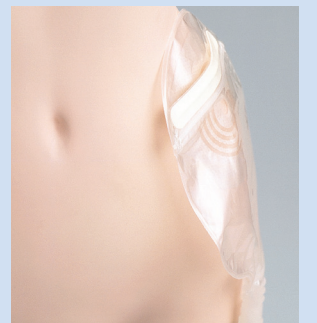

A flat and compact zipper system for opening / closing fits the body well and minimizes discomfort during wearing.

Drainage after Operation ● EOG Sterilized



SURGIDRAIN OPEN-TOP

Protection of Wounds by Dome-Shaped Cap (retractable): High-performance Drainage Pouch, in Variable Sizes

- Wounds can be observed and treated by opening the cap, while the pouch is still attached.
- The skin barrier is soft and flexible, with little residual paste and excellent fitting.
- A soft-type cap allows viscous excrement to be drained by milking from outside.
- SURGIDRAIN OPEN-TOP Post-Op-M features a back-flow prevention valve.



With the retractable cap, wounds can be observed and treated, while the pouch is still attached.



The dome-shaped cap protects wounds from external pressure.


Drainage after Operation ● EOG Sterilized




WELLCARE DRAIN

The special drainage pouch for fistula has a skin barrier with high pH buffering action.

- The thin and flexible skin barrier for fistula offers excellent pH buffering action.
- A soft-type outlet allows viscous excrement to be drained by milking from outside.
- Drainage pouches come in 3 sizes for various types of fistula.

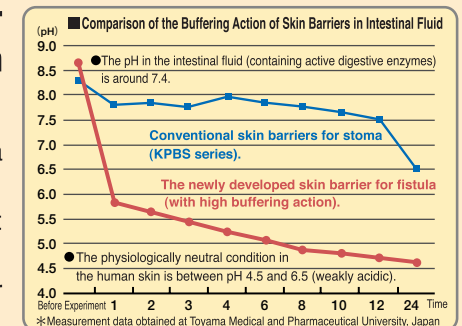

Fistula Drainage

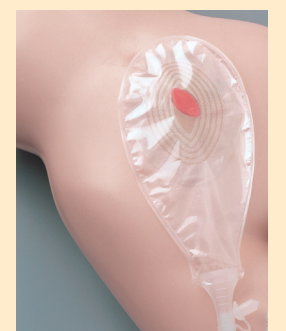

Skin irritation caused by drainage is prevented due to the high buffering action of the skin barrier.