


Youcare Ostomy System



Feature 1

Assurance for Skin Friendly

Skin-Friendly Skin Barrier of Natural Materials

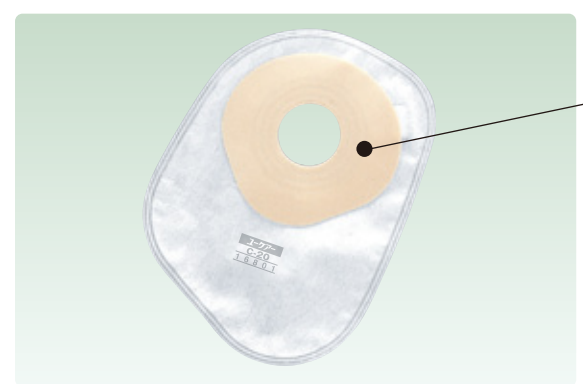


The main ingredients of the skin barrier are the natural materials that rarely irritate the skin such as "karaya gum" and "citrus pectin". Karaya gum keeps the surface of the faceplate weak acid.


Feature 2

Easy and Simple Handling

User-Friendly Clip-less Ostomy Appliances

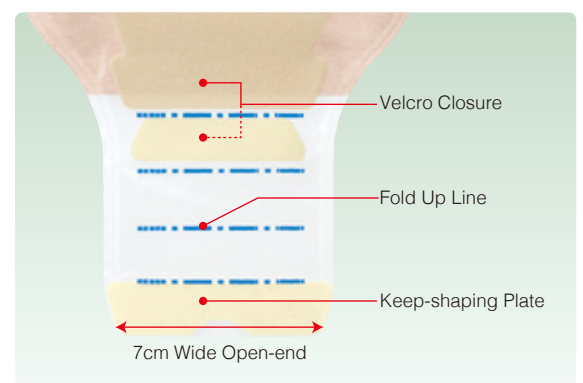

Just turn up the plate along with the blue fold up line and secure the end with Velcro closures attached. Opening and closing are easy. The keep-shaping plate makes discharge and wiping easy.

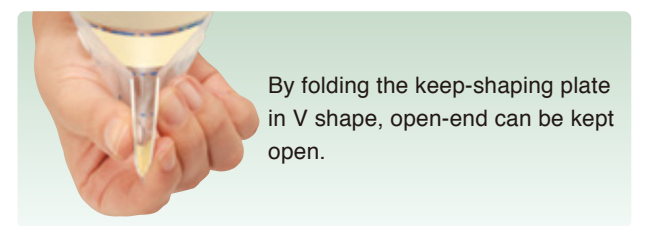


By folding the keep-shaping plate in V shape, open-end can be kept open.

Feature 3

Comfortable Application

Scientific Analysis of Body Movement Diamond Shape

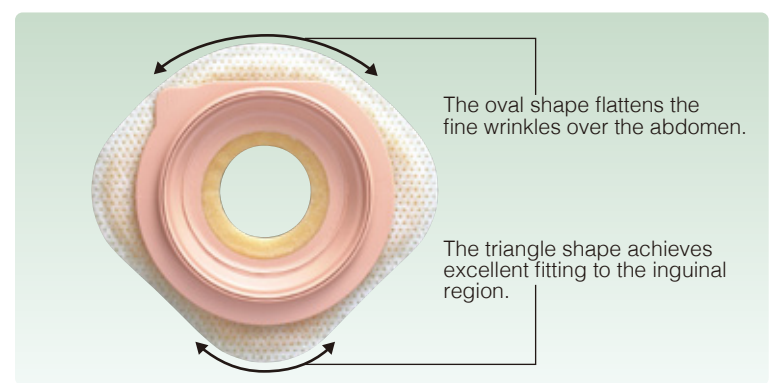


Anatomically designed Diamond Shape effectively prevents detachment and leakage by increased flexibility following the curved groin and the abdominal movement. Because of the compact design, he/she hardly notices uncomfortable feeling. It offers higher levels of durability and achieves excellent contact.

Youcare Ostomy System

Digestive System Ostomy Appliances

1-Piece Ostomy System


Youcare-D 2 Days Exchange



Open-ended Clip Flat

Type	Product Code No.	Starter Hole Size	Max Available Size	Bag Size (Length X Width)	Contents Per Box
Flesh	16761	20mmφ	14-59mm (free-cut)		30pcs
Without Filter	16771	20mmφ	14-59mm (free-cut)		
20	16781	20mmφ	14-59mm (free-cut)		
25	16782	25mmφ	19-24mm	275mm X 150mm	
30	16783	30mmφ	24-29mm		
35	16784	35mmφ	29-34mm		
40	16785	40mmφ	34-39mm		
45	16786	45mmφ	39-44mm		
50	16787	50mmφ	44-49mm		


Youcare-Dc 2 Days Exchange



Open-ended Clip Convex

Type	Product Code No.	Starter Hole Size	Max Available Size	Bag Size (Length X Width)	Contents Per Box
22	16811	22mmφ	18-21mm		10pcs
25	16812	25mmφ	21-24mm		
28	16813	28mmφ	24-27mm		
32	16814	32mmφ	27-31mm	275mm X 150mm	
36	16815	36mmφ	31-35mm		
40	16816	40mmφ	35-39mm		
45	16817	45mmφ	39-44mm		
50	16818	50mmφ	44-49mm		


Youcare-TD 2 Days Exchange



Open-ended Clips Flat

Type	Product Code No.	Starter Hole Size	Max Available Size	Bag Size (Length X Width)	Contents Per Box
Flesh	17441	20mmφ	14-59mm (free-cut)		10pcs
Without Filter	17451	20mmφ	14-59mm (free-cut)		
20	17461	20mmφ	14-59mm (free-cut)		
25	17462	25mmφ	19-24mm	275mm X 150mm	
30	17463	30mmφ	24-29mm		
35	17464	35mmφ	29-34mm		
40	17465	40mmφ	34-39mm		
45	17466	45mmφ	39-44mm		
50	17467	50mmφ	44-49mm		


Youcare-TDc 2 Days Exchange



Open-ended Clips Convex

Type	Product Code No.	Starter Hole Size	Max Available Size	Bag Size (Length X Width)	Contents Per Box
22	17471	22mmφ	18-21mm		10pcs
25	17472	25mmφ	21-24mm		
28	17473	28mmφ	24-27mm		
32	17474	32mmφ	27-31mm	275mm X 150mm	
36	17475	36mmφ	31-35mm		
40	17476	40mmφ	35-39mm		
45	17477	45mmφ	39-44mm		
50	17478	50mmφ	44-49mm		


Youcare-C 1 Day Exchange



Closed Flat

Type	Product Code No.	Starter Hole Size	Max Available Size	Bag Size (Length X Width)	Contents Per Box
Flesh	16791	20mmφ	14-59mm (free-cut)		30pcs
20	16801	20mmφ	14-59mm (free-cut)		
25	16802	25mmφ	19-24mm		
30	16803	30mmφ	24-29mm	200mm X 150mm	
35	16804	35mmφ	29-34mm		
40	16805	40mmφ	34-39mm		
45	16806	45mmφ	39-44mm		
50	16807	50mmφ	44-49mm		

Youcare-Cc 1 Day Exchange




Closed Convex

Type	Product Code No.	Starter Hole Size	Max Available Size	Bag Size (Length X Width)	Contents Per Box
22	16821	22mmφ	18-21mm		10pcs
25	16822	25mmφ	21-24mm		
28	16823	28mmφ	24-27mm		
32	16824	32mmφ	27-31mm	200mm X 150mm	
36	16825	36mmφ	31-35mm		
40	16826	40mmφ	35-39mm		
45	16827	45mmφ	39-44mm		
50	16828	50mmφ	44-49mm		

2-Piece Ostomy System


Youcare 2-F 3 to 5 Days Exchange

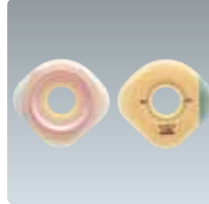

Flat

Type	Product Code No.	Starter Hole Size	Max Available Size	Face Plate Size	Applicable Bag	Contents Per Box
M22	16591	22mmφ	16-34mm (free-cut)	97mmX105mm	M	5pcs
M25	16592	25mmφ	21-24mm	97mmX105mm	M	
M28	16593	28mmφ	24-27mm	97mmX105mm	M	
L32	16594	32mmφ	27-31mm	108mmX117mm	L	
L36	16595	36mmφ	31-35mm	108mmX117mm	L	
L40	16596	40mmφ	35-39mm	108mmX117mm	L	
L45	16597	45mmφ	4-44mm (free-cut)	108mmX117mm	L	
65	16598	10mmφ	4-64mm (free-cut)	129mmX140mm	65	

Youcare 2-Fc 3 to 5 Days Exchange




Convex

Type	Product Code No.	Starter Hole Size	Max Available Size	Face Plate Size	Applicable Bag	Contents Per Box
M22	16601	22mmφ	18-21mm	110mmφ	M	5pcs
M25	16602	25mmφ	21-24mm	110mmφ	M	
M28	16603	28mmφ	24-27mm	110mmφ	M	
L32	16604	32mmφ	27-31mm	120mmφ	L	
L36	16605	36mmφ	31-35mm	120mmφ	L	
L40	16606	40mmφ	35-39mm	120mmφ	L	

2-Piece Ostomy System


Youcare 2-D



Open-ended Clip

Type	Product Code No.	Specification Bag Size (Length X Width)	Contents Per Box
Clear	M 16621	300mmX150mm	10pcs
	L 16622	300mmX150mm	
	65 16623	300mmX150mm	
Flesh	M 17751	300mmX150mm	10pcs
	L 17752	300mmX150mm	


Youcare 2-Df



Open-ended Clip

Type	Product Code No.	Specification Bag Size (Length X Width)	Contents Per Box
Clear	M 16611	300mmX150mm	10pcs
	L 16612	300mmX150mm	
	65 16613	300mmX150mm	
Flesh	M 17741	300mmX150mm	10pcs
	L 17742	300mmX150mm	


Youcare 2-TD



Open-ended Clips

Type	Product Code No.	Specification Bag Size (Length X Width)	Contents Per Box
Clear	M 17781	300mmX150mm	10pcs
	L 17782	300mmX150mm	
	65 17783	300mmX150mm	
Flesh	M 17791	300mmX150mm	10pcs
	L 17792	300mmX150mm	


Youcare 2-TDf



Open-ended Clips

Type	Product Code No.	Specification Bag Size (Length X Width)	Contents Per Box
Clear	M 17761	300mmX150mm	10pcs
	L 17762	300mmX150mm	
	65 17763	300mmX150mm	
Flesh	M 17771	300mmX150mm	10pcs
	L 17772	300mmX150mm	


Youcare 2-Cf



Closed

Type	Product Code No.	Specification Bag Size (Length X Width)	Contents Per Box
Clear	M 16631	210mmX150mm	10pcs
	L 16632	210mmX150mm	
	M 17801	210mmX150mm	
Flesh	L 17802	210mmX150mm	10pcs

Youcare 2-M




Closed

Type	Product Code No.	Specification Bag Size (Length X Width)	Contents Per Box
M	16671	120mmX130mm	10pcs
	16672	120mmX130mm	

Urology System Ostomy Appliances

1-Piece Ostomy System


Youcare-U 2 to 3 Days Exchange



Outlet for Urine Flat

Type	Product Code No.	Starter Hole Size	Max Available Size	Bag Size (Length X Width)	Contents Per Box
10	15361	10mmφ	4-59mm (free-cut)		10pcs
20	15362	20mmφ	14-19mm		
25	15363	25mmφ	19-24mm	260mm X 150mm	
30	15364	30mmφ	24-29mm		
35	15365	35mmφ	29-34mm		
40	15366	40mmφ	34-39mm		

Youcare-Uc 2 to 3 Days Exchange




Outlet for Urine Convex

Type	Product Code No.	Starter Hole Size	Max Available Size	Bag Size (Length X Width)	Contents Per Box
13	16401	13mmφ	9-12mm		10pcs
16	16402	16mmφ	12-15mm		
19	16403	19mmφ	15-18mm		
22	16404	22mmφ	18-21mm	260mm X 150mm	
25	16405	25mmφ	21-24mm		
28	16406	28mmφ	24-27mm		
32	16407	32mmφ	27-31mm		

Urology System Ostomy Appliances

2-Piece Ostomy System


Youcare 2-Fu 4 to 7 Days Exchange



Flat

Type	Product Code No.	Starter Hole Size	Max Available Size	Face Plate Size	Applicable Bag	Contents Per Box
M13	16641	13mmφ	7-34mm (free-cut)	97mmX105mm	M	5pcs
M16	16642	16mmφ	12-15mm	97mmX105mm	M	
M19	16643	19mmφ	15-18mm	97mmX105mm	M	
M22	16644	22mmφ	18-21mm	97mmX105mm	M	
M25	16645	25mmφ	21-24mm	97mmX105mm	M	
M28	16646	28mmφ	24-27mm	97mmX105mm	M	
L32	16647	32mmφ	27-44mm (free-cut)	108mmX117mm	L	


Youcare 2-Fuc 4 to 7 Days Exchange

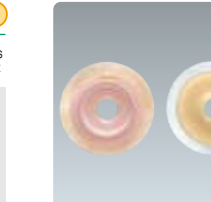

Convex

Type	Product Code No.	Starter Hole Size	Max Available Size	Face Plate Size	Applicable Bag	Contents Per Box
M13	16651	13mmφ	9-12mm	110mmφ	M	5pcs
M16	16652	16mmφ	12-15mm	110mmφ	M	
M19	16653	19mmφ	15-18mm	110mmφ	M	
M22	16654	22mmφ	18-21mm	110mmφ	M	
M25	16655	25mmφ	21-24mm	110mmφ	M	
M28	16656	28mmφ	24-27mm	110mmφ	M	
L32	16657	32mmφ	27-31mm	120mmφ	L	

Youcare 2-U




Outlet for Urine

Type	Product Code No.	Specification Bag Size (Length X Width)	Contents Per Box
M	16661	260mmX150mm	10pcs
	16662	260mmX150mm	


Accessories

Youcare Trunk Belt




Type	Product Code No.	Waist Size	Applicable Bag	Contents Per Box
—	15091	60-110mm	Youcare 2-TDf	1Pce
			Youcare 2-TD	
			Youcare 2-Df	
			Youcare 2-D	
			Youcare 2-Cf	
			Youcare 2-U	
			Youcare 2-IS	
			Youcare 2-M	
			Youcare 2-BC	
			Youcare-TDc	
Youcare-Dc				
Youcare-Cc				
Youcare-Uc				
Ileline D-Cap				
Ileline TD				
Ileline D				

Youcare Trunk Belt With Face Plate




Type	Product Code No.	Waist Size	Applicable Bag	Contents Per Box
—	15081	60-110mm (diameter of hole 73mmφ)	Youcare-TD Youcare-D Youcare-C Youcare-U Pospack-B Pospack-Light	1set

Youcare 2-BC



Type	Product Code No.	Specification (Height of Dome)	Contents Per Box
M	16681	20mm	2pcs
L	16682	20mm	2pcs
65	16683	20mm	2pcs

Youcare 2-IS



Type	Product Code No.	Specification Bag Size (Length X Width)	Contents Per Box
M	16691	900mmX100mm	5pcs
	16692	900mmX150mm	