

HN1 & HN2 TEST

Institut de Bourgogne – Dijon



<Challenge>

Context:

Stereotactic treatment of a lesion at the elbow

Problems:

Elbow bend management for reproducibility

Patient comfort at the lesion

<Application>

Installation of a Moldcare under the elbow with the addition of a simple foam to rest the wrist.

Picture of Application



<Result>

CT Simulation:

Easy to use

A small sprayer to moisten the Moldcare would have been a plus.

Dosimetry:

Low density material, low impact of restraint mitigation.

Treatment:

Comfortable

On this treatment (only 3 sessions): Good rigidity & Reproducible arm bend.

On the other hand, for this treatment, wrist rotation is not managed by the Moldcare.