


## Instructions for use



Medical device name : Multi Fix-Roll

Medical device description : Semi-permeable Waterproof Film



### [Appearance]

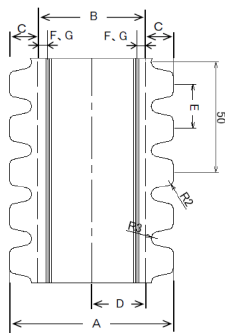

Type : No.5 , No.7

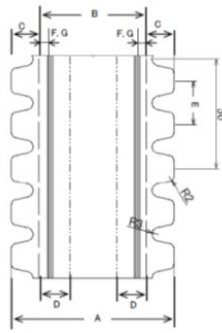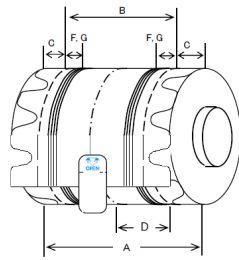


Type : No.10 , No.15

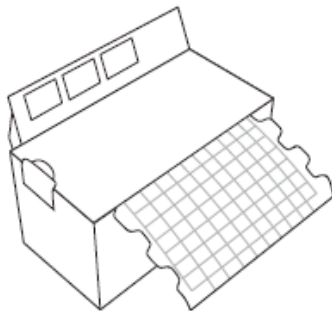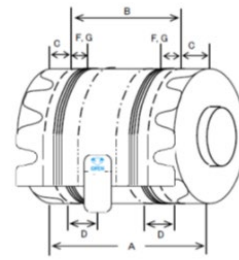

Type : No.5 , No.7 , No.10 , No.15

### [Cross section diagram]

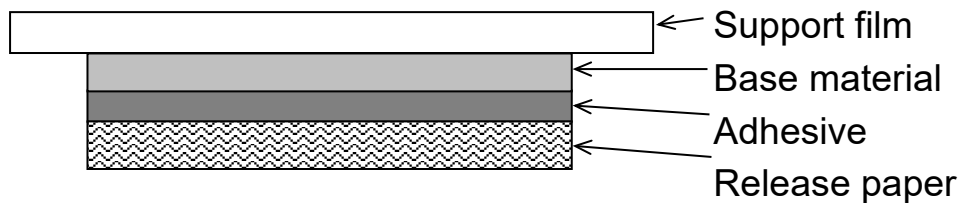

[Examples of use]

Absorbent (e.g., dressing). Not included in product.



Fixing



Waterproofing



Friction and Slippage

[Product size]

Commodity code	Type	Specification		Quantity per box
		Width [mm]	Length [m]	
17821	No.5	50	10	1 roll
17822	No.7	75		1 roll
17823	No.10	100		1 roll
17824	No.15	160		1 roll

[Product color]


	Color
Tape	Transparent
Support film	Transparent

Intended use : Fixing, Prevention from dirt, Waterproofing,  
Friction and Slippage prevention


1. Instructions for use:

[Type : No.5, No.7]

- ① Peel off the one side of the release paper from the center and attach it to the skin.

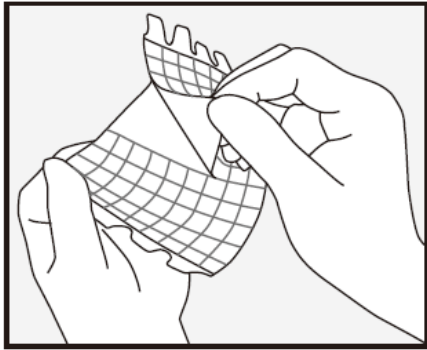

- ② Peel off the remaining release paper to stick it well entirely to the skin.

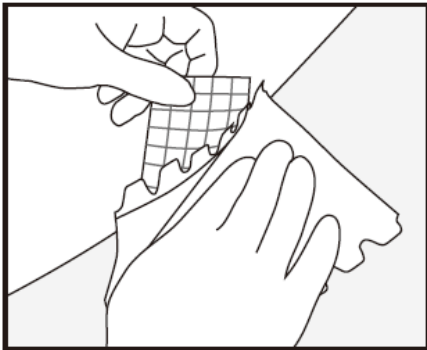


- ③ Pinch the side tabs and remove the support film.




[Type : No.10, No.15]


① Peel off the center release paper and attach it to the skin.

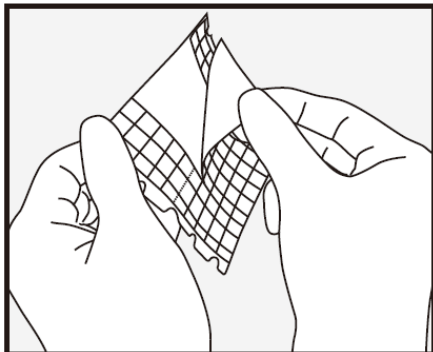

② Peel off the remaining release paper to stick it well entirely to the skin.

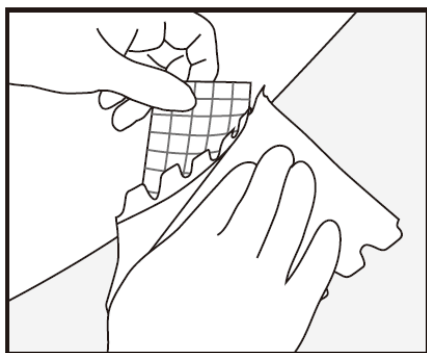

③ Pinch the side tabs and remove the support film.

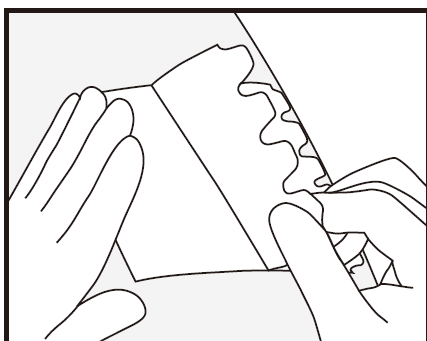



## 2. Precautions for use:

- This product is not sterilized. Do not apply directly to the wound or puncture site.
- Observe the application site as suitable during application.  
If you experience rash, redness, itchiness, etc., discontinue its use and consult a doctor or pharmacist.
- When peeling off this product, hold the skin with one hand and pinch a corner of the film in order not to deform the skin, and slowly pull and peel it off parallel to the skin along the flow of hair. If it is difficult to peel it off, use a remover.
- If the product floats or peels off during application, replace it.
- Do not apply excessive strain on the skin. It may cause skin damage.
- When using this product for waterproofing purposes such as showers, make sure that the film adheres well without any gaps in advance.
- Please note that it may be difficult to adhere depending on the material of the equipment or when the surface of the skin is oily or wet.
- Dispose of in accordance with all applicable local regulations.
- Single use only

## 3. Storage condition:

- Store in a cool and dark place to avoid getting wet, direct sunlight, high temperature and humidity.

4. SYMBOLS:

SYMBOLS	
	Expiry Date
	Lot number

Expiry Date : Do not use beyond this date.

Lot number : Manufacturing identification number

=====

[Importer information]

Importer : Marubeni Thailand Co., Ltd.

Address : 548 One City Centre Building, 32nd- 33rd Floor, Ploenchit Road,  
Lumpini, Pathumwan, Bangkok 10330

Phone : 02 256 6900

=====

=====

[Manufacturer information]

Manufacturer : ALCARE Co., Ltd.

Address : 19F Arcacentral Bldg., 1-2-1 Kinshi, Sumida-ku, Tokyo, 130-0013, Japan

=====

- Prior to use, carefully read the instruction for use so that you understand the product's features.
- Keep the instruction for use close at hand and read it again when needed.