


Instructions for use

Medical device name : Presnet

Medical device description : Elastic Mesh Bandage

[Appearance]

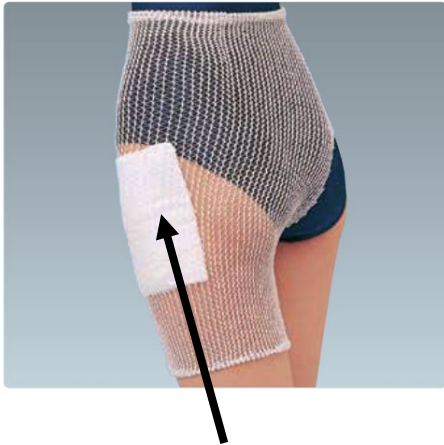

Do not include absorption dressing.

[Product size]

Commodity code	Type	Specification		Quantity per box
		Width [mm] (Reference value)	length [m]	
11671	No.1	7	25	1 roll
11672	No.1.5	10		1 roll
11673	No.2	16		1 roll
11674	No.3	32		1 roll
11675	No.4	42		1 roll
11676	No.5	50		1 roll
11677	No.6	62		1 roll
11678	No.7	75		1 roll

[Product color]

* White

Intended use : Cover the affected area. (Part fixed with gauze or sprint)

1. Special features

- Easy and speedy bandage treatment is possible just by covering.
- With excellent elasticity, it fits every part.
- A wide variety of types can be used for all parts from the fingers to the trunk.



2. Precautions for use

- Pay attention to abnormalities such as pain and numbness in the wearing part and peripheral part, congestion, and skin disorders such as rash and itch, and check accordingly. Also, instruct the patient to contact a doctor immediately if they feel any of these abnormalities.
- Dispose of in accordance with all applicable local regulations.
- Single use only.

3. Storage condition

- Avoid getting wet
- Store at room temperature, avoiding direct sunlight and high temperature and humidity.

4. SYMBOLS:

SYMBOLS	
	Expiry Date
	Lot number

Expiry Date : Do not use beyond this date.

Lot number : Manufacturing identification number

=====

[Importer information]

Importer : Marubeni Thailand Co., Ltd.
Address : 548 One City Centre Building, 32nd- 33rd Floor, Ploenchit Road,
Lumpini, Pathumwan, Bangkok 10330
Phone : 02 256 6900

=====

=====

[Manufacturer information]

Manufacturer : ALCARE Co., Ltd.
Address : 19F Arcacentral Bldg., 1-2-1 Kinshi, Sumida-ku, Tokyo, 130-0013, Japan

=====

- Prior to use, carefully read the instruction for use so that you understand the product's features.
- Keep the instruction for use close at hand and read it again when needed.