


## Instructions for use

Medical device name : SI-Aid

Medical device description : Silicone-Faced Wound Dressing

[Appearance] & [Cross section diagram]

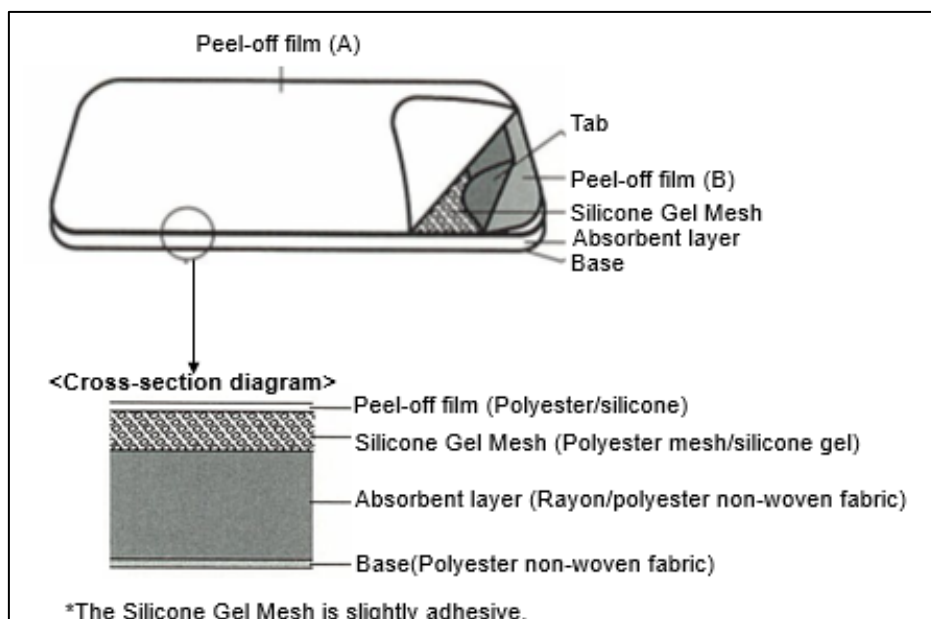

[Product size]

Commodity code	Type	Specification (width x length)	Round cut	Quantity per box
18751	No.3	75mm x 75mm	With	50 pieces
18752	No.5	100mm x 100mm	With	50 pieces
18753	No.7	100mm x 200mm	With	30 pieces
18754	No.10	200mm x 300mm	Without	10 pieces
18755	No.11	300mm x 550mm	Without	5 pieces

[Product color]

\* White

Intended use : Cover and protection for wounds, burns, etc. and absorption of exudate

## 1. Contraindications / Prohibitions

- Reuse prohibited

## 2. Shape / Structure

- This product is a wound dressing protective material for the purpose of cover and protection for wounds, burns, etc. and absorption of exudate.
- The gel layer has adhesiveness and improves the operability of this product and the adhesion to the affected area during application.

## 3. Principle

This product, which is composed of an absorption layer and a gel layer, protects the coating by attaching it to wounds, burns, etc.

In addition, exudate, blood, etc. pass through the mesh-like gel layer and are absorbed.

## 4. Instructions for use

### 4-1. Before application

- (1) Cleanse the application site of this product and disinfect it as needed.
- (2) Dry the peri-wound area sufficiently.
- (3) Select the appropriate size enough to cover the whole wound area.

\*Cut the dressing to the desired shapes if necessary.

### 4-2. Application

- (1) Hold the tabs of the release film (white) and release film (blue) of this product and peel off the films, avoid touching the application surface and then apply it to the wound.
- (2) If the exudate is heavy, another sheet of this product or sterilized absorptive gauze can be used by overlaying.
- (3) Fix this product with an appropriate means such as adhesive tape or bandage, etc.

### -Precautions-

- This product does not have a waterproof function.
- Fix this product with waterproof tape etc. when you have a shower or bath.

### 4-3. Replacement

- (1) Replace it when necessary, considering the amount of exudate and the wound condition.
- (2) Remove this product slowly while pressing the skin.
- (3) If this product sticks to the wound surface, moisten the adhered part with distilled water or saline to soften it and then peel it off.

## 5. Precautions for use

### 5-1. Important basic precautions

- (1) Discontinue use of this product and take appropriate measures if any symptoms considered skin disorders such as redness, erosion and itching appear during use.
- (2) If there is a clinical infection on the wound surface, or if there is a risk of it, observe it frequently and take appropriate measures.

Also, pay careful attention in use especially when the health condition is bad.

[The symptoms may get worse.]

### 5-2. Defects / adverse events

[Defects / Adverse events] in the use of general wound dressings / protective materials

- Infectious symptoms of wound
- Wound and peri-wound skin disorders (epidermal detachment, maceration, edema, blister, redness, erosion, itching, allergy, contact dermatitis).
- Fixation
- Increase of necrotic tissue
- Pain

### 5-3. Other precautions

- (1) Do not use it if the sterilized packaging is damaged or soiled.
- (2) Use this product immediately after opening the sterilized packaging.
- (3) Remove it carefully and gently when used on skin grafts.



[The transplanted skin may be peeled off when removed.]

- (4) Dispose of in accordance with all applicable local regulations.

## 6. Storage condition

- Avoid getting wet.
- Store at room temperature, avoiding direct sunlight and high temperature and humidity.

## 7. SYMBOLS:

SYMBOLS	
	Expiry Date
	Lot number

Expiry Date : Do not use beyond this date.

Lot number : Manufacturing identification number

=====

### [Importer information]

Importer : Marubeni Thailand Co., Ltd.

Address : 548 One City Centre Building, 32nd- 33rd Floor, Ploenchit Road,  
Lumpini, Pathumwan, Bangkok 10330

Phone : 02 256 6900

=====

=====

### [Manufacturer information]

Manufacturer : ALCARE Co., Ltd.

Address : 19th floor, Arcacentral Building, 1-2-1 Kinshi, Sumida-ku, Tokyo, 130-0013, Japan

=====

- Prior to use, carefully read the instruction for use so that you understand the product's features.
- Keep the instruction for use close at hand and read it again when needed.