小児用プロケア 1 Pediatric Procure 1

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- Prior to use, carefully read the instruction manual so that you understand the product's features.
- Keep the instruction manual close at hand and read it again when needed.

Application Illustration is Drainable Type

1 Clean the Skin Around the Stoma.
Before you apply an ostomy bag, softly clean the skin around the stoma with gauze which is soaked with lukewarm water, and dry well. Use soap to thoroughly remove any oil from the surface of the skin as it may weaken adhesion.

2 Make a Hole in the Cutting Gauge.
Measure your stoma size and make the stoma size hole in the cutting gauge. Make sure the hole comes center of the gauge.

3 Make a Hole on the Skin Barrier.
Place the cutting gauge on the release paper of the skin barrier. Trace the hole onto the skin barrier. Cut the skin barrier 1 to 2mm larger than the indicated line.

4 Peel Off the Release Paper.
Do not cut a larger hole than the maximum available diameter, or it may cause a leakage.

5 Apply the Ostomy Bag to the Stoma.
Apply the skin barrier to fit the stoma while checking the ostomy site.

6 Hold the Adhesion Part Well.
Hold the peristomal skin well by hands to adhere well to the skin.

How to Close Open-end (Post-OP/Drainable Type)

1 Turn up the open-end at about 3, 4cm from the end.

2 Turn up again at 1cm width like the illustration.

3 Fold the both sides in three times.

4 Again fold the both sides in three times, and secure by a rubber band at about 1.5cm from the end.

How to Use the Uro Products Connecting Tube (Urinary Type)

1 Use the attached connecting tube for connection to a urine collection bag. Insert the connector of the urine collection bag into the vinyl tube part of the connecting tube.

2 Squeeze the connector part of the connecting tube into the urinary outlet of the ostomy bag until you hear a click.

3 To remove the connector, pull out while pressing down the ring part.

Precautions for Use

- Do not apply anything around the stoma. Reduced adhesion may be a cause of leakage.
- Do not touch the adhesive face with oil-covered hands. Reduced adhesion may be a cause of leakage.
- Do not reapply an ostomy bag once it has been peeled off. Reduced adhesion may be a cause of leakage.
- Carefully cut a hole in the skin barrier, making a hole beyond the specified cut line may result in leakage through the skin barrier.
- When the adhesive surface of the skin barrier is cold, warm it and then apply.
- When you find any skin trouble, stop using immediately.

Precautions for Stock

- Do not store at the following conditions which may decrease the adhesive quality to cause leakage:
Excessively humid place Having direct sunlight
Excessively cold place Having physical pressure
Excessively hot place
- Do not store the skin barrier with release paper removed. Reduced adhesion may be a cause of leakage.

Prior to use

The customers are advised to receive guidance from their treating physicians or nurses before using this product. They are also advised to carefully read the instructions and fully understand the use of this product in advance. Erroneous handling may result in leakage of faeces and the development of leakage related dermatitis. As the skin of the children is particularly delicate, softly remove from the skin using soap. Stop using this product if you notice any skin irritation.

Types of Pediatric Procure 1

Post-OP	Drainable Type	Urinary Type
Immediately after Operation (Use for Colostomy/Ileostomy)	After Discharge from Hospital (Use for Colostomy/Ileostomy)	After Discharge from Hospital (Use for Urostomy)
For the sensitive skin immediately after operation, a skin-friendly "Karaya Plast" skin barrier is used.	A durable skin barrier is used for adhesion. For further secure required, a skin-friendly "Karaya Plast" skin barrier (ALCARE Code No.14441) is available.	

Precautions for Use

- Do not apply anything around the stoma. Reduced adhesion may be a cause of leakage.
- Do not touch the adhesive face with oil-covered hands. Reduced adhesion may be a cause of leakage.
- Do not reapply an ostomy bag once it has been peeled off. Reduced adhesion may be a cause of leakage.
- Carefully cut a hole in the skin barrier, making a hole beyond the specified cut line may result in leakage through the skin barrier.
- When the adhesive surface of the skin barrier is cold, warm it and then apply.
- When you find any skin trouble, stop using immediately.

Precautions for Stock

- Do not store at the following conditions which may decrease the adhesive quality to cause leakage:
Excessively humid place Having direct sunlight
Excessively cold place Having physical pressure
Excessively hot place
- Do not store the skin barrier with release paper removed. Reduced adhesion may be a cause of leakage.

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