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- Before use, carefully read the instruction manual so you understand the product's features.
- Keep this instruction manual close at hand and refer to it again when needed.

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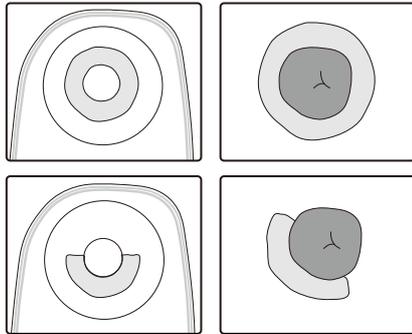
## Application 用途

### Ring type 环型

Convenient for reinforcing and adjusting the entire circumference of the stoma. You can spread the hole diameter to the stoma size and apply it directly.  
You can also use it for partial adjustment by cutting out the required amount.

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## Introduction 引言

"Cellcare-Soft Wafer" is a skin barrier for ceramide compounding that can be molded by hands, and has outstanding adhesion and durability. The product is available in ring and stick types. The product fills wrinkles and hollows around the stoma, increasing adherence of the stoma appliance and protecting the skin from excrement. For your safety, do not use it for any purpose other than the intended use, and use it according to this instruction manual.

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### Warning

This product is not food. To prevent accidental ingestion of the product, manage it appropriately, such as by keeping it out of reach of elderly people, people with dementia, infants and children.

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### Precautions

- Do not apply ointment or other products to the skin around the stoma, as this may lead to reduced adhesion and subsequent leakage.
- Wash and dry the skin around the stoma well before use. Dirt, oil, or soap residue left on the skin may cause leakage due to reduced adhesion.
- When the product is cold, it may be hard to apply to the skin. In this case, move the product to a warm room and let the whole product warm up before use.
- Do not reuse the product once removed from the skin. Reuse may cause leakage due to reduced adhesion.
- Do not use the product beyond its expiry date. Using expired product may cause leakage due to reduced adhesion.
- In the unlikely event that the product reacts with the skin, stop treatment immediately and consult a physician or a nurse.

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### How to apply

- Take out the required amount from the tray.  
\*Cellcare-Soft Wafer can be cut with scissors or torn by hand.  
\*Return surplus Cellcare-Soft Wafer to the tray and store after covering it.
- Peel off the release paper/film on both sides of the Cellcare-Soft Wafer.
- Adjust the Cellcare-Soft Wafer to appropriate size and shape.
- Apply the adjusted Cellcare-Soft Wafer to the face plate of the stoma appliance or skin. After attaching the stoma appliance as usual, press it with your hands so it fits comfortably to the skin, and ensure that it adheres well.

### How to remove

You can peel off Cellcare-Soft Wafer together with the face plate.  
\*If it adheres to the skin and is hard to peel off, please use a peeling agent.

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## Precautions for storage 保管上的注意事项

Follow the storage instructions below to avoid quality deterioration such as insufficient adhesion.

- Do not store at high temperature ( $\geq 40^{\circ}\text{C}$ ) and high humidity.
- Do not store at low temperature (e.g., in a refrigerator).
- Do not store in direct sunlight.
- Avoid long-term storage due to bulk purchase.  
\*Please be sure to check the expiry date printed on the box.
- Do not store under pressure.
- Do not store outside the tray.
- Do not peel off the release paper/film until use.
- Do not store in any way other than flat stacking.

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## How to dispose of the product 废弃处理注意事项

Similar to the stoma appliance, wrap the product with newspaper and place it in a garbage bag to dispose of it after use. The appliance is usually regarded as "non-burnable waste," but the method for separating garbage varies by region. Please contact your local authorities for more information.

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