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## Mini Pouch

洗腸後パウチ  
Post-Irrigation Pouch

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Post-Irrigation Pouch

## Instruction Manual

- Prior to use, carefully read the instruction manual so that you understand the product's features.
- Keep the instruction manual close at hand and read it again when needed.

You can contact us at  
E-mail : inquiry@alcare.co.jp Fax : +81-3-5611-7827  
www.alcare.co.jp/e

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

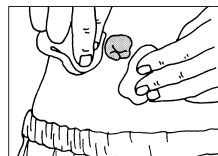
## How to Apply (These pictures show Mini Pouch-K Super)

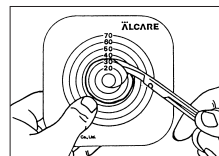
①Clean the Skin around the Stoma  
Clean the skin around the stoma with gauze which is soaked with lukewarm water. Dry well. Use bath soap to remove any oil from the surface of the skin thoroughly as it may weaken its adhesion.

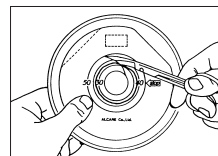
②Make a Hole in the Cutting Gauge  
Measure the stoma size. Make a hole as the same size as the stoma in the cutting gauge. Maximum size of the hole should be within 50mm, and the hole should be made in the center.

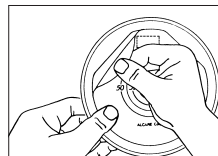
③Make a Hole in the Skin Barrier  
Place the cutting gauge on the release paper of the skin barrier. Trace the hole onto the release paper with a pen. Cut out the hole with scissors etc.

④Peel Off the Release Paper

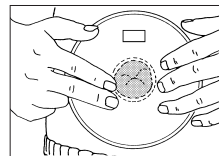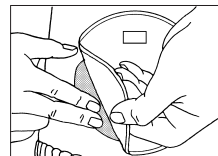
⑤Apply to Fit the Stoma  
Apply the faceplate to fit the stoma while checking the ostomy position.

⑥Press Down the Adhesion Area  
Press the adhesion area against the skin around the stoma to ensure adhesion.

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
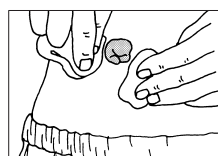
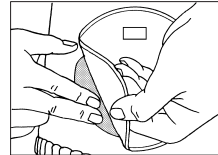
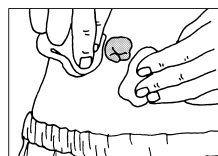
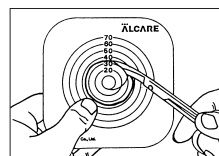
## How to Use the Air Release Filter (Mini Pouch-K Super)

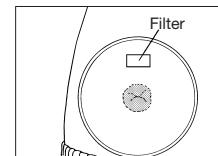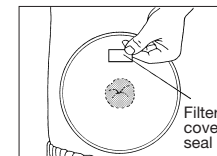①The accumulated air inside the ostomy bag is gradually released through the filter.

②Normally, put the filter cover seal on the filter. When gas has accumulated inside the ostomy bag, peel the filter cover seal to release it.

## Precautions for Use and Storage

- Do not apply ointment or cream to the skin around the stoma.
- Do not touch the adhesive face with oil-covered hands.
- Do not reapply an ostomy bag once it has been peeled off.
- Avoid long-term stockpiling.
- Do not peel off the release paper until use.
- Do not make a hole beyond the specified cut line.
- Do not store in an area of high-temperature or high-humidity area or in a freezer.
- Do not store in direct sunlight area and avoid any pressure.
- Warm the adhesive face before application if it is cold.
- Stop using if you have any skin trouble.

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## ALCARE Co., Ltd.